

# EQUIA Forte™ Technique Guide



1. Occlusal caries.



2. Cavity preparation.



3. Apply CAVITY CONDITIONER (10 sec.) or DENTIN CONDITIONER (20 sec.).



4. Rinse, dry; do not desiccate.



5. EQUIA Forte™ Fil is placed in one increment.



6. Shape and contour within 60 sec.



7. Final finish after 2 min 30 sec.



8. Brush-apply a single layer of EQUIA Forte Coat.



9. Light-cure for 20 sec.



10. Completed restoration.



### **EQUIA Forte™ Fil**

Box: 50 capsules

Shades: A1, A2, A3, A3.5, B1,  
B2, B3, C4

Assorted (contains 10 each  
A2, A3, A3.5, B1, B3)

### **EQUIA Forte™ Coat**

Bottle 4mL

## **EQUIA Forte™ Glass Hybrid Restorative System**

- Bulk placement in less than 5 minutes from start to finish
- No bonding agent needed – chemical adhesion with tooth structure
- Low moisture sensitivity
- Optimal protection of marginal seal for long lasting restorations
- Great shade-matching ability
- High lustre, translucent coating, saving you polishing time



GC Australasia Dental Pty Ltd  
1753 Botany Road Banksmeadow  
NSW 2019 Australia

**T:** +61 2 9301 8200  
**E:** [info@gcaustralasia.com](mailto:info@gcaustralasia.com)  
**[www.gcaustralasia.com](http://www.gcaustralasia.com)**