

# seT



(seT capsules)

SELF ETCHING, SELF ADHESIVE RESIN CEMENT

SELBSTÄTZENDER, SELBSTKLEBENDER KUNSTSTOFF-ZEMENT

CIMENTO RESINOSO AUTO-CONDICIONANTE E AUTO-ADESIVO

CEMENTO DE RESINA, AUTOGRABANTE Y AUTOADHESIVO

RÉSINE DE SCELLEMENT AUTO-ADHÉSIVE AUTO-MORDANÇANT

CEMENTO COMPOSITO AUTOMORDENZANTE ED AUTOADESIVO

ZELF ETSEND, ZELF ADHESIEF RESIN CEMENT

SELVÆTSENDE, SELVADHÆRERENDE RESIN CEMENT

SELVETSENDE, SELVBINDENDE RESIN SEMENT

ITSE-ETSAAVA, ITSEKIINNITTYVÄ MUOVISEMENTTI

ΑΥΤΟΑΔΡΟΠΟΙΟΥΜΕΝΗ, ΑΥΤΟΣΥΓΚΟΛΛΟΥΜΕΝΗ ΡΗΤΙΝΩΔΗΣ ΚΟΝΙΑ

SAMOWYTRAWIAJĄCY CEMENT KOMPOZYTOWY NIE WYMAGAJĄCY UŻYCIA SYSTEMU WIĄŻĄCEGO

ÖNSAVAZÓ ÉS ÖNKÖTŐ MŰGYANTABÁZISÚ RAGASZTÓCEMENT

ISESÖÖVITAV, ISEADHESEERUV TSEMENT

SAMOJEDKAJOČ, SAMOADHEZIVEN SMOLNAT CEMENT

PAŠKODINOŠS, PAŠADHEZĪVS SVEĶU CEMENTS

SAVAIME ĒSDINANTIS BEI SURIŠANTIS DERVŪ CEMENTAS

SAMOLEPTACÍ, ADHEZIVNÍ CEMENT ZESÍLENÝ PRYSKYŘICÍ

SAMOLEPTACÍ, ADHEZÍVNÝ, ŽIVICOVÝ CEMENT

歯科接着用レジンセメント セルフエッチングデュアルキュアタイプ

自酸蚀、自粘结树脂粘结剂





# SDI's top rated cement, seT now available in syringes



seT




seT is a self etching, self adhesive, resin cement, designed for final cementation of metal, resin, all ceramic and ceramic based restoratives. seT chemically bonds to tooth structure and all types of core material. seT is available in capsules and a paste/paste automix syringe system (seT PP).

**Proprietary ionglass™ filler**  
seT utilises SDI's proprietary ionglass™ filler developed by our glass technologists. ionglass™ is a radiopaque, high ion releasing, reactive glass used in SDI's range of dental cements. seT releases substantially higher fluoride to assist with remineralisation of the natural dentition.

### High fluoride release




seT's free movement of fluoride provides benefits to the tooth. Fluoride plays several significant roles in any caries - prevention program. These include the formation of fluorapatite, which is more acid resistant than hydroxyapatite. Fluorapatite is harder than other tooth minerals. It helps to provide protection from tooth decay.

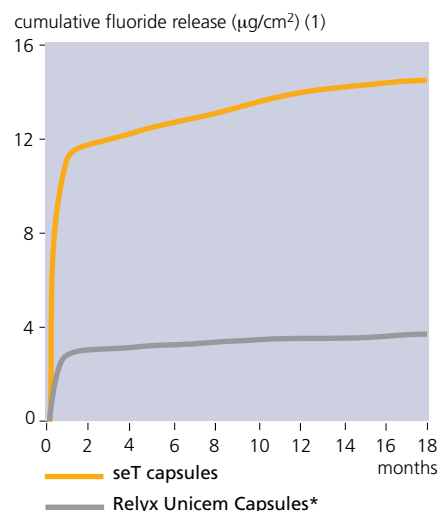

## Fast and easy

seT is an all in one luting cement. It etches, bonds and fills with just the one component. Simply apply seT and seat your restoration.

seT produces a low film thickness and fluidity to seat cemented restorations.

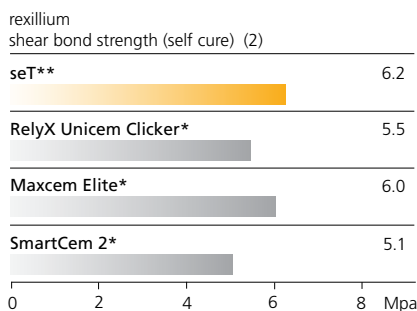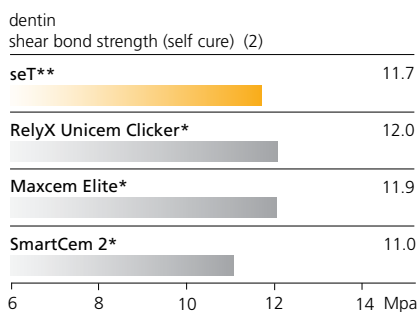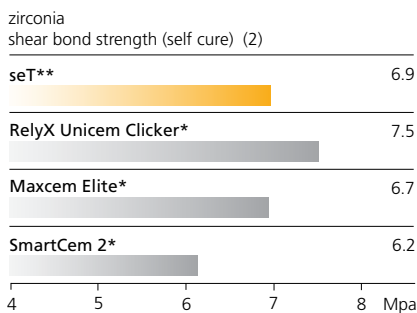

## No Bisphenol A or HEMA

seT does not contain any Bisphenol A (including its derivatives) or HEMA. Use this product on your patients with confidence and peace of mind.

## Dual cure

Designed to combine the advantages of light curing and self curing, seT is the ideal luting cement for most of your cementation needs<sup>^</sup>.

## Radiopacity

Detectable on x-ray radiographs, seT is great for recalls when margins are regularly checked for integrity.

## Minimal to no post-operative sensitivity

seT does not require separate etching and washing. The smear layer remains intact to protect the tooth. This dramatically reduces, if not eliminates, post-operative sensitivity.

## Choice of delivery tips

seT PP syringes can be adapted for a variety of delivery options. The regular mixing tips are for predominantly used for dispensing into crown and bridge. In conjunction with a wide mixing tip, endodontic and intraoral tips are also available.



## Extensive colour range

seT is available in 5 shades: Translucent, A1, A2, OA3 and Opaque White. The large variety of shades helps maximise aesthetics.

## Indications

- Cementation of metal, resin, all ceramic and ceramic based inlays, onlays<sup>^</sup>, bridges and crowns.
- Cementation pins, posts and screws

<sup>^</sup> In the case of veneers, use an adhesive (Go!, Stae, etc according to manufacturer's instructions including any etching procedure) prior to seT to improve the retentive strength. Not suitable for veneers requiring extended working time.

\*\*seT PP.

## instructions:

- 1 Remove temporary cement residue and rinse thoroughly. Remove excess water, keep moist.



- 2 Prepare restoration according to manufacturer's instructions, and check fit.



- 3 Extrude cement evenly onto all bonding surfaces of the restoration and tooth.



- 4 Seat restoration, and leave for 30 seconds. Tack cure excess for 2 seconds or wait 2 minutes.



- 5 Trim excess material away.



- 6 Maintain isolation until set is verified (approximately 5 minutes), or light cure through translucent restoration for at least 20 seconds.



# seT



### seT PP automix syringe refill

2 seT PP 7g automix syringes  
28 automix syringe mixing tips

Reorder:

8805101 A1  
8805102 A2  
8805103 OA3  
8805104 White Opaque  
8805105 Translucent

### seT PP assorted automix syringe kit

1 each of A1, A2, OA3, White Opaque and Translucent  
7g seT PP automix syringes  
70 automix syringe mixing tips  
Reorder 8810002

### seT PP automix syringe mixing tips

50 automix syringe mixing tips  
Reorder 8810003

### seT capsules assorted kit

10 each of A1, A2, OA3, White Opaque & Translucent  
seT capsules  
Reorder: 8810000

### seT capsules

50 seT capsules  
Reorder:  
8805001 A1  
8805002 A2  
8805003 OA3  
8805004 White Opaque  
8805005 Translucent

### 50 x seT PP Mixing Tips - Regular

Reorder: 8810003

### 10 x seT PP Mixing Tips - Wide

Reorder: 8810004

### 10 x seT PP Oral Tips

(To be used in conjunction with seT PP Mixing Tips - Wide)  
Reorder: 8810005

### 10 x seT PP Endo Tips

(To be used in conjunction with seT PP Mixing Tips - Wide)  
Reorder: 8810006



**Riva Applicator 2<sup>+</sup>**  
Reorder 5545013



**Riva Applicator<sup>+</sup>**  
Reorder 5545009

\*Not the registered trademarks of SDI limited.

(1) Source: Al-Naimi O.T., McCabe J. F. Fluoride release of seT self-etch/self-adhesive resin cement with that of Competitor Products. (18 months report) Newcastle University, U.K. February 2007.

(2) SDI Test Data.

seT: THE DENTAL ADVISOR, Vol. 25, No. 08. October 2008

seT PP: THE DENTAL ADVISOR, Vol. 27, No. 7, Sept. 2010, THE DENTAL ADVISOR preferred product 2011.



Made in Australia by SDI Limited

Bayswater, Victoria 3153

Australia 1 800 337 003

Austria 00800 022 55 734

Brazil 0800 770 1735

France 00800 022 55 734

Germany 0800 100 5759

Ireland 01 886 9577

Italy 800 780625

New Zealand 0800 734 034

Spain 00800 022 55 734

United Kingdom 00800 022 55 734

USA & Canada 1 800 228 5166

www.sdi.com.au

